

"瑞邦" 補強粘合片

"Ribbond" Bondable Reinforcement

衛署醫器輸字第 022226 號

使用前請務必詳閱原廠之使用說明書並遵照指示使用

◆ 產品敘述及用途

單一缺牙牙橋

適用於 Ribbond 與支台樁緊密連結在一起，且放置在牙齒切緣中間的 pontic 區域。在 pontic 區域使用兩段 Ribbond。

如果空間允許，對於臨時牙橋的準備並非必要性。如果沒有足夠的空間，需要有一點準備措施。對於長期的牙橋，準備措施應該要足夠的讓複合材料塗抹於 Ribbond 上。

● 注意事項及警告用語：

1. 不要使用會侵蝕性的樹脂。我們也建議不要使用包含象牙質的一大罐接合系統。
2. Ribbond 沒有很光滑。不要切除 Ribbond 纖維。
3. 光治療夾板。治療單顆牙齒一次需要 30-40 秒。
4. 在使用樹脂之前，不要使用手或乳膠手套觸碰 Ribbond。請使用聶子、鉗子、棉線手套。

◆ 型號及規格

Catalog No.	Description	Contain
Ribbon Bondable Reinforcement Ribbon		
AST	Assorted Starter Kit	One scissors, one tin foil, and one 2mm x 22 cm, one 3mm x 22 cm, one 4mm x 22 cm Ribbon.
KIT2	2 mm Starter Kit	One scissors, one tin foil, and one 2 mm x 68 cm Ribbon.
KIT3	3 mm Starter Kit	One scissors, one tin foil, and one 3mm x 68 cm Ribbon.
KIT4	4 mm Starter Kit	One scissors, one tin foil, and one 4mm x 68 cm Ribbon.
KITO	Orthodontic Starter Kit	One scissors, one tin foil and one Ortho x 22 cm Ribbon.
REAST	Assorted Refill	One tin foil and one 2mm x 22 cm, one 3mm x 22 cm, one 4mm x 22 cm Ribbon.
RE2	2 mm Refill	One tin foil and one 2 mm x 68 cm Ribbon.
RE3	3 mm Refill	One tin foil and one 3mm x 68 cm Ribbon.
RE4	4 mm Refill	One tin foil and one 4mm x 68 cm Ribbon.
RE9	9 mm Refill	One tin foil and one 45 cm x 9 mm Ribbon.
Ribbon THM (Thinner Higher Modulus)		
ASTTHM	Assorted Starter Kit - THM	One scissors, one tin foil, and one 2mm x 22 cm, one 3mm x 22 cm, one 4mm x 22 cm Ribbon TH-
KIT2THM	2 mm Starter Kit - THM	One scissors, one tin foil, and one 2 mm x 68 cm Ribbon THM.
KIT3THM	3 mm Starter Kit - THM	One scissors, one tin foil and one 3mm x 68 cm Ribbon THM.
KIT4THM	4 mm Starter Kit - THM	One scissors, one tin foil, and one 4mm x 68 cm Ribbon THM.
KIOTHM	Orthodontic Starter Kit -THM	One scissors, one tin foil and one Ortho x 68 cm Ribbon THM.
REASTTHM	Assorted Refill -THM	One tin foil and one 2mm x 22 cm, one 3mm x 22 cm, one 4mm x 22 cm Ribbon THM.
RE2THM	2 mm Refill -THM	One tin foil and one 2 mm x 68 cm Ribbon THM.
RE3THM	3 mm Refill -THM	One tin foil and one 3mm x 68 cm Ribbon THM.
RE4THM	4 mm Refill -THM	One tin foil and one 4mm x 68 cm Ribbon THM.
REOTHM	Ortho Refill -THM	One tin foil and one Ortho x 68 cm Ribbon THM.
Ribbon Triaxial		
ASTTRI	Assorted Triaxial Starter Kit	One scissors, one tin foil, two Triaxial Thin Ribbon and two Triaxial Dense Ribbon.
REASTTRI	Assorted Triaxial Refill	One tin foil and two Triaxial Thin Ribbon and two Triaxial Dense Ribbon.

製造商：Ribbon, Inc.

廠址：1904 3rd AVE Ste 330Seattle, WA 98101, USA

藥商名稱：杏昌生技股份有限公司

藥商地址：新北市三重區重新路五段609巷2號8樓之2